

TEE TRANSPORT TRAY

Simplify Your Workflow and Stay Compliant with the TEE Transport Tray



Simplify your TEE probe cleaning process by contacting **800.445.6741** to order your **TEE Transport Tray**

REF #	PRODUCT DESCRIPTION	QTY
610-1418	TEE Probe Transport Tray	30
610-1471	TEE Transport Tray (90) and Enzymatic Sponge (100) Bundle	90

Compliant Workflow

Reversible lid for easy **clean/dirty** indication

Protect Your Investment

Keep your expensive, delicate probe safe from damage by protecting the distal tip

Prevent Cross-Contamination

Safeguard disinfected probe shaft from the handle, cord and connector

OEM Approved

Meet probe manufacturer transducer coiling requirements

Simplified Process

Eliminate time and resources needed to clean reusable tray

Stay Clean, Be Green

Made from recycled materials



CIVCO.com/TEE-Transport | info@civco.com | 800.445.6741

COPYRIGHT © 2018. ALL RIGHTS RESERVED. CIVCO IS A REGISTERED TRADEMARK OF CIVCO MEDICAL SOLUTIONS. ALL PRODUCTS MAY NOT BE LICENSED IN ACCORDANCE WITH CANADIAN LAW. PRINTED IN USA. 2018R-3058 REV 2

TEE TRANSPORT TRAY WORKFLOW

Use the same tray to transport to and from the exam room with ease



- 1** Stack clean trays near probe storage. Transport clean probe in clean tray to exam room.
- 2** Post-exam, place dirty probe in same tray & flip lid to dirty. Transport probe to reprocessing location and dispose of tray.
- 3** Use a clean tray to transport the disinfected probe to its storage cabinet.

Visit [CIVCO.com/TEE-Transport](https://www.civco.com/TEE-Transport) to view the TEE Transport Tray workflow video



CIVCO.com | info@civco.com | 800.445.6741

